Adhere

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(Adhesive)

This hoof adhesive is used for hoof wall repairs, building heel angles, and gluing on steel, aluminum, and polyurethane shoes. Adhere provides ample working time but is weight bearing in just four minutes.

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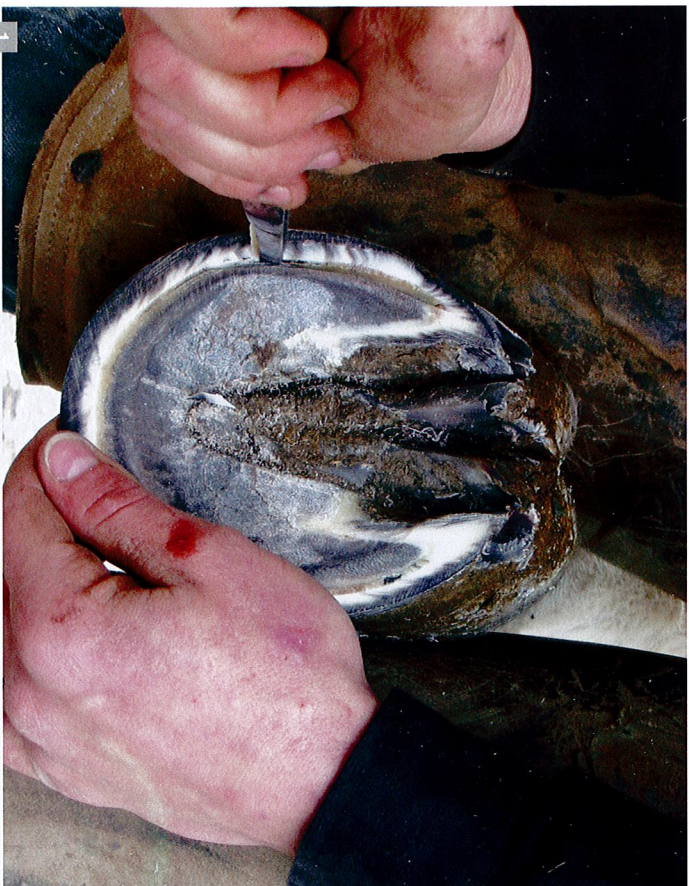
Adhere is a fast setting urethane adhesive which provides a superior bond to steel, aluminum, and most synthetic shoes.

As with all glue on procedures, foot and shoe prep are vital.

Adhere provides a great alternative when poor wall condition or lameness prevents nailing.

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Remove anything loose, flaky, stelly or diseased.

For maximum results avoid bonding to loose or flaky hoof wall. It is important to bond to the strongest parts of the hoof.

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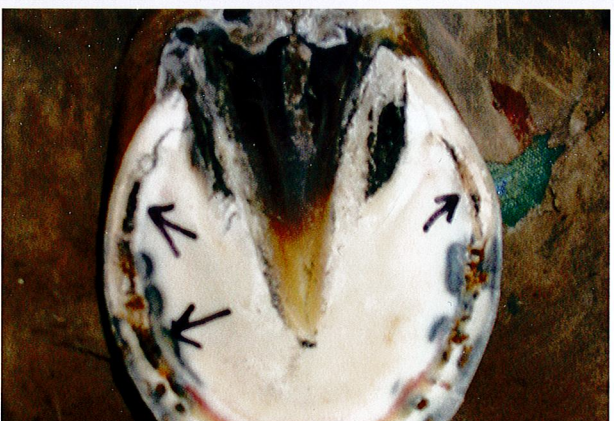
REMEMBER

Adhere will stick, just make sure it's to the correct surface, the strongest surface, you have on the hoof.

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NOTE

The arrows above. Generally when gluing on, areas of separation exist around the white line. This loose, flaky area must be removed as it will compromise the adhesive bond and may contain anaerobic bacteria (which should not be covered).

The area around the bars should also be removed down to good solid structure.

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NOTE

The loose flaky areas have been relieved and only solid hoof structure remains. Once the area around the bars is cleaned, this hoof is ready to be dried and the shoe applied.

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REMEMINDER

Adhering to good, solid hoof structure is optimum.

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Shoe prep is as vital as hoof prep.

Both steel and aluminum shoes should be ground to reveal fresh, new metal. All oxidation should be removed. Use clips, this handles the shear factor. Enlarge nail holes, drill at heels if gluing there. Provide as much mechanical hold as possible. The clips should be 95% set to the hoof before gluing. Adhere should not be between the clip and the hoof wall.

Vettec Adhesive Spacers provide an even thickness of adhesive around the entire shoe, which is critical.

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Spacers (in both photos above) provide 3 mm. thickness.

Ref.	Beskrivning / Description
46025	adhesive spacers

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DRY, DRY, DRY

Drying the hoof is essential. Adhere will bond to the first surface it touches, even water!

Whether you're in the Northeast or the desert Southwest, that hoof retains moisture. As soon as you start trimming the hoof and cutting the horn tubules, you're releasing moisture. Dry it!

When you're done drying, it's time to glue the shoe on. If you must put the foot down, do so with a sock on it or onto a clean towel.

Left alone too long the hoof will rehydrate itself and too much moisture will be present.

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Dispense Adhere onto the shoe as quickly as possible to a depth of about 15mm to 20mm. This is not the time to be neat. Therefore, use gloves.

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Set shoe to hoof until clips stop you, then push down until the Spacers hit the hoof. At this point the shoe should not move. With a finger, start at one heel and smooth the material on the sole to the next heel.

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Wrapping hoof in pallet wrap is suggested. This ensures no slippage.

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NOTE

Adhere coming through the enlarged nail holes. Hold for 4 minutes. Set down.

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(Left picture) Extra material at heels is added to prevent movement.

If choosing to leave heels "free", place tape on shoe at heel but still put adhesive on shoe in that area. This will allow the material to stick to the hoof but not the shoe providing a level surface.

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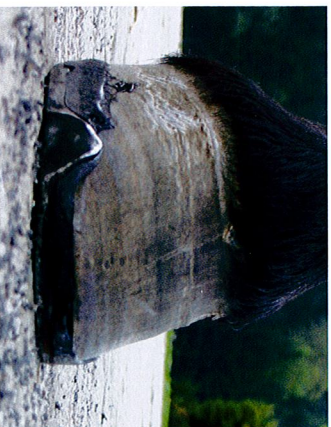
For working horses, apply extra material up onto the hoof wall and heels for extra support.

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
In any type of glue on procedure, getting one nail on each side will greatly increase the life of the glue on shoe.

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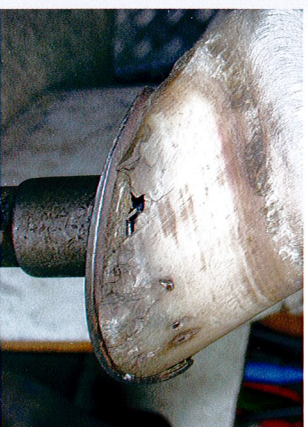Adhere is the clear choice for:

- Filling in cosmetic chips and cracks.
- Building hoofwall for support.
- Rebuilding hoofwall for nailing on shoes.

The benefits of Adhere are:

- No fumes or odors.
- Fast Setting.
- Handles like hoof for nailing, rasping, and nipping.
- Available in Beige or Black.

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When resecting and fluid is visible: **STOP! DO NOT REPAIR.** Wait 7-10 days and then proceed.

If the hoof has not been treated with ointments or oils, **DO NOT** use a solvent to clean. When using a cleaner, **NEVER USE ACETONE**, use denatured alcohol.

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NOTE

All weak and flaky wall has been removed. Undercuts can be used if desired. Creating mechanical locks could extend the life of the repair. Outer periole layer of hoof wall should be rasped or sanded lightly to allow solid bond to "new" hoof structure.

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Because a dremel tool was used to prep the hoof (creating a lot of dust) use a clean towel with denatured alcohol and wipe clean. Do not use a squeeze bottle and flood the area. This can leave a film and inhibit a solid bond.

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Dry the hoof!

Even in a dry climate or if the hoof appears dry, the hoof still retains moisture. Once the horn tubules are cut moisture is released.

In this case alcohol was used as a cleaning agent. The alcohol will sit in the tubules until it is air dried. It is important to adhere to the hoof, not the alcohol or moisture.

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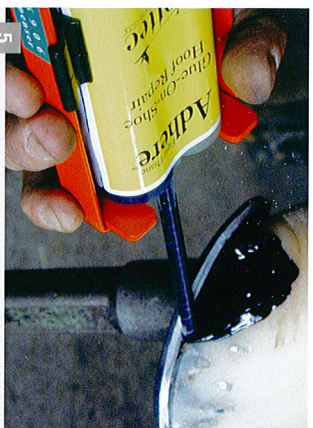
After drying, place mix tip against repair area and dispense using circular motions.

To keep the material from sticking to the shoe, apply some tape to the shoe before nailing.

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Overfill material to extend beyond the prepped area. Leave approximately 3 mm to 4 mm thickness.

Do not "play" with the material, mold or "feather" it. Leave it full.

NOTE

Black Adhere was used for visual contrast. Normally beige Adhere would be used for cosmetic purposes.

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Adhere will set hard to the touch in 45 - 60 seconds.

Final set is approximately 6 - 8 minutes.

Adhere can repair hoof defects on feet with or without shoes.

If using shoes, it is recommended that tape is applied to the shoe where the repair will be. This will prevent the Adhere from sticking to the shoe.

If the horse pulls the shoe, the repair will not be pulled from the wall.

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The Adhere is at least 3 - 4 mm thick while setting. This provides a faster and harder set.

If not going into the show ring, rasp smooth but not thin. Leave it full as this provides increased strength.

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Wait until all the heat has dissipated before working the material. Again, rasp smooth but not thin, more thickness means more strength.

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After finishing, nail and clinch. It is not necessary to "set" the nail in first and then fill with Adhere. Adhere takes a nail and clinches just like natural hoof wall.

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This repair has been "feathered" for show. More Adhere may be added to strengthen the repair.

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NOTE:

The material covers more than the prepared area. This will help with the compression, bending, and expansion phases of the hoof.

Black Adhere was used for visual contrast. Normally beige Adhere would be used.

Adhere

Vettec
Vettec